

# Keep GJ tubes clear with the TubeClear® system.



## GJ-1422 for use with the following GJ Tubes:

Actuated Medical's latest TubeClear Clearing Stem Model GJ-1422, is designed to help clear AMT G-Jet®, Avanos MIC® and MIC-KEY® Gastro-jejunosotomy (GJ) feeding tubes\*. These tubes feature multiple channels within the tube, creating some distinct differences for the user when approaching with the TubeClear system compared to other feeding tubes.

G-Jet®, MIC®, and MIC-KEY® Gastro-jejunosotomy Tubes that are size 14 - 22 Fr and have a jejunal length of 15 - 45 cm (6 - 18 in).



### Coconut Oil

GJ-1422 is supplied with Coconut Oil for lubrication during the clearing procedure. Coconut Oil lubrication is key to success in GJ tubes. Reapply prior to each Clearing Stem insertion.



### Shorter Motion

When using GJ-1422 to clear a GJ tube, it's important that your backward and forward motion is shorter – about 2 cm (1 in) from the tube's port – to prevent kinking the Clearing Stem.



### Longer Time to Clear

Given the complex nature of GJ tubes, it can take longer for GJ-1422 to work compared to other tubes – sometimes up to an hour.

**~30**  
minutes or  
less to clear<sup>1</sup>



MANUFACTURED BY: Applied Medical Technology, Inc. (AMT) Brecksville, OH

**~60**  
minutes or  
less to clear<sup>2,3</sup>



MANUFACTURED BY: Avanos Medical, Inc. (formerly Halyard Health and Kimberly-Clark®) Alpharetta, GA

 **TubeClear®**

Enteral Access Clearing System

 Learn more at  
[TubeClear.com](http://TubeClear.com)

FDA Clearance #:  
K200646, K172556, K163092, K131052, K123659, K121571

\* Please review the TubeClear Operator's Manual for full indications, contraindications, warnings, and cautions.

**Rx ONLY**

