


# GJ-1422 Depth Limiter Set-up Example for MIC® GJ Tubes.

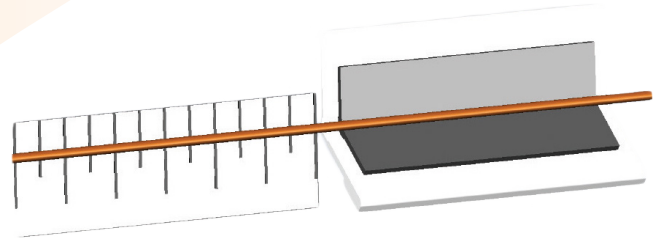

## MIC® GJ Tube Depth Limiter Placement Example

**TubeClear®**  
Enteral Access Clearing System

Tube Manufacturer:  
**Avanos Medical, Inc.**

Tube Type: **MIC® GJ Tube**

1. Tube Jejunal Length: 30 cm
2. Tube's French Size: 16 Fr
3. Port to be Cleared: Jejunal (J)
4. Place Depth Limiter at 61 cm

Taken From the TubeClear Operator's Manual **TABLE 14**  
MIC® GJ Tube Depth Limiter Placement Chart

Circle colors coordinate with the appropriate instructions from example (left)

Jejunal Length (cm)	French (Fr)	Depth Limiter Placement (cm)	
		Jejunal (J) Port	Gastric (G) Port
15	16 Fr	46	32
22	16 Fr 18 Fr	53	32
30 <sup>1</sup>	16 Fr <sup>2</sup> 18 Fr	61 <sup>3</sup>	32
45	16 Fr 18 Fr 22 Fr	76	



**TubeClear®** Learn more at [TubeClear.com](http://TubeClear.com)

Enteral Access Clearing System FDA Clearance #: K200646, K172556, K163092, K131052, K123659, K121571

\* This does not substitute for the requirement to read the TubeClear Operator's Manual.



1100793760-000 **Rx ONLY**  
Pat. ActuatedMedical.com/ip