


GJ-1422 Depth Limiter Set-up Example for G-Jet® GJ Tubes.



Taken From the TubeClear Operator's Manual **TABLE 16**
G-Jet® GJ Tube Depth Limiter Placement Chart

Circle colors coordinate with the appropriate instructions from example (left)

G-Jet® GJ Tube Depth Limiter Placement Example



Enteral Access Clearing System

Tube Manufacturer:
Applied Medical Technology, Inc.

Tube Type: **G-Jet® GJ Tube**

1. Tube Jejunal Length: 22 cm
2. Tube's French Size: 18 Fr
3. Tube's Stoma (cm): 1.2 cm
4. Port to be Cleared: Gastric (G)
5. Place Depth Limiter at 5.5 cm

Jejunal Length (cm)	French (Fr)	Stoma (cm)	Depth Limiter Placement (cm)	
			Jejunal (J) Port	Gastric (G) Port
15	14 Fr	1.0	20	5
		1.2	20.5	5.5
		1.5	20.5	5.5
		1.7	21	6
22 ¹	14 Fr	1	27	5
		1.2 ³	27.5	5.5 ⁵
	16 Fr	1.5	27.5	5
		1.7	28	6
		2.0	28	6
		2.3	28.5	6.5
30	14 Fr	1.2	35.5	5.5
		1.5	35.5	5.5
		1.7	36	6
		2.0	36	6
		2.3	36.5	6.5
		2.5	36.5	6.5
		2.7	37	7
45	14 Fr	1.5	50.5	5.5
		1.7	51	6
		2.0	51	6
		2.3	51.5	6.5
		2.5	51.5	6.5
		2.7	52	7
		3.0	52	7
		3.5	52.5	7.5
		4.0	53	8
		4.5	53.5	8.5
		5.0	54	9
		5.5	54.5	9.5
		6.0	55	10

