


GJ-1422 Depth Limiter Set-up Example for MIC-KEY® GJ Tubes.

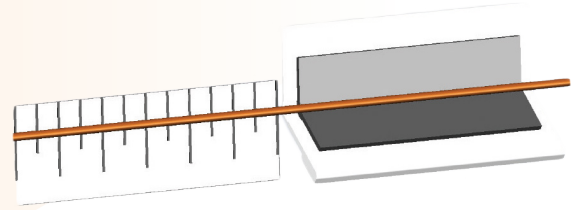

Taken From the TubeClear Operator's Manual **TABLE 15**
MIC-KEY® GJ Tube Depth Limiter Placement Chart

Circle colors coordinate with the appropriate instructions from example (left)

MIC-KEY® GJ Tube Depth Limiter Placement Example

TubeClear
Enteral Access Clearing System

Tube Manufacturer:
Avanos Medical, Inc.

Tube Type: **MIC-KEY® GJ Tube**

1. Tube Jejunal Length: 45 cm
2. Tube's French Size: 14 Fr
3. Tube's Stoma (cm): 2.3 cm
4. Port to be Cleared: Jejunal (J)
5. Place Depth Limiter at 53 cm

Jejunal Length (cm)	French (Fr)	Stoma (cm)	Depth Limiter Placement (cm)	
			Jejunal (J) Port	Gastric (G) Port
15	14 Fr 16 Fr	1.0	21.5	6.5
		1.2	22	7
		1.5	22	7
		1.7	22.5	7.5
22	14 Fr 16 Fr 18 Fr	1	28.5	6.5
		1.2	29	7
		1.5	29	7
		1.7	29.5	7.5
		2.0	29.5	7.5
30	14 Fr 16 Fr 18 Fr	1.2	37	7
		1.5	37	7
		1.7	37.5	7.5
		2.0	37.5	7.5
		2.3	38	8
		2.5	38	8
		2.7	38.5	8.5
45	14 Fr 16 Fr 18 Fr 22 Fr	1.5	52	7
		1.7	52.5	7.5
		2.0	52.5	7.5
		2.3	53	8
		2.5	53	8
		2.7	53.5	8
		3.0	53.5	8
		3.5	54	8
		4.0	54.5	9.5
		4.5	55	10
		5.0	55.5	10.5
		5.5	56	10.5
		6.0	56.5	11
6.5	57	12		
7.0	57.5	12.5		



FDA Clearance #: K200646, K172556, K163092, K131052, K123659, K121571



Intertek



1100793764-000

Pat. ActuatedMedical.com/ip

Rx ONLY

* This does not substitute for the requirement to read the TubeClear Operator's Manual.