

## Effects of clogged feeding tubes on syringe force - a bench top analysis

Maureen L. Mulvihill, Ph.D., Olga M. Ocón-Grove, Ph.D., Eric M. Steffan, Abigail M. DeLellis, Kiara Yough Weltner,  
and Paula M. Mulhall, RPN, RGN, RN  
Actuated Medical, Inc., 320 Rolling Ridge Dr., Bellefonte, PA 16823

In an 18Fr enteral tube, material build-up of 1.3 mm on the inner Tube walls will decrease the diameter to the equivalent of a 14Fr Tube, thus decreasing flow rate by as much as 50%.

Scan here to  
learn more

### Introduction


- + Feeding tubes (Tubes) are life-sustaining; providing nutrition, medication, and hydration to those unable to ingest orally.
- + Tubes clog at rates up to 35%<sup>1-3</sup>, resulting in nutrition and medication interruptions.
- + Some clogs can form quickly, while others occur over time, causing uncertainty amongst healthcare practitioners regarding a patient's Tube patency status.
- + Research discusses preventative measures to Tube clogging and ways to remove clogs; however, the question as to how one can determine if a Tube is fully patent does not appear to have been addressed.

*How does one ascertain when material build-up is adhering to the interior Tube wall if fluids can still be introduced into the Tube?*

- + **Study Objectives:** Quantify the amount of required force to introduce air into Tubes with various levels of clogging with the end goal to correlate those values with a subjective “feel.”

### Methods

- + Within 14Fr Nasoenteric (NE) Tubes, in vitro produced clogs were created using a mixture of feeding formula and fiber.
- + Three (3) Tube Conditions were tested, N=15 trials, with n=5 trials/Tube Condition:

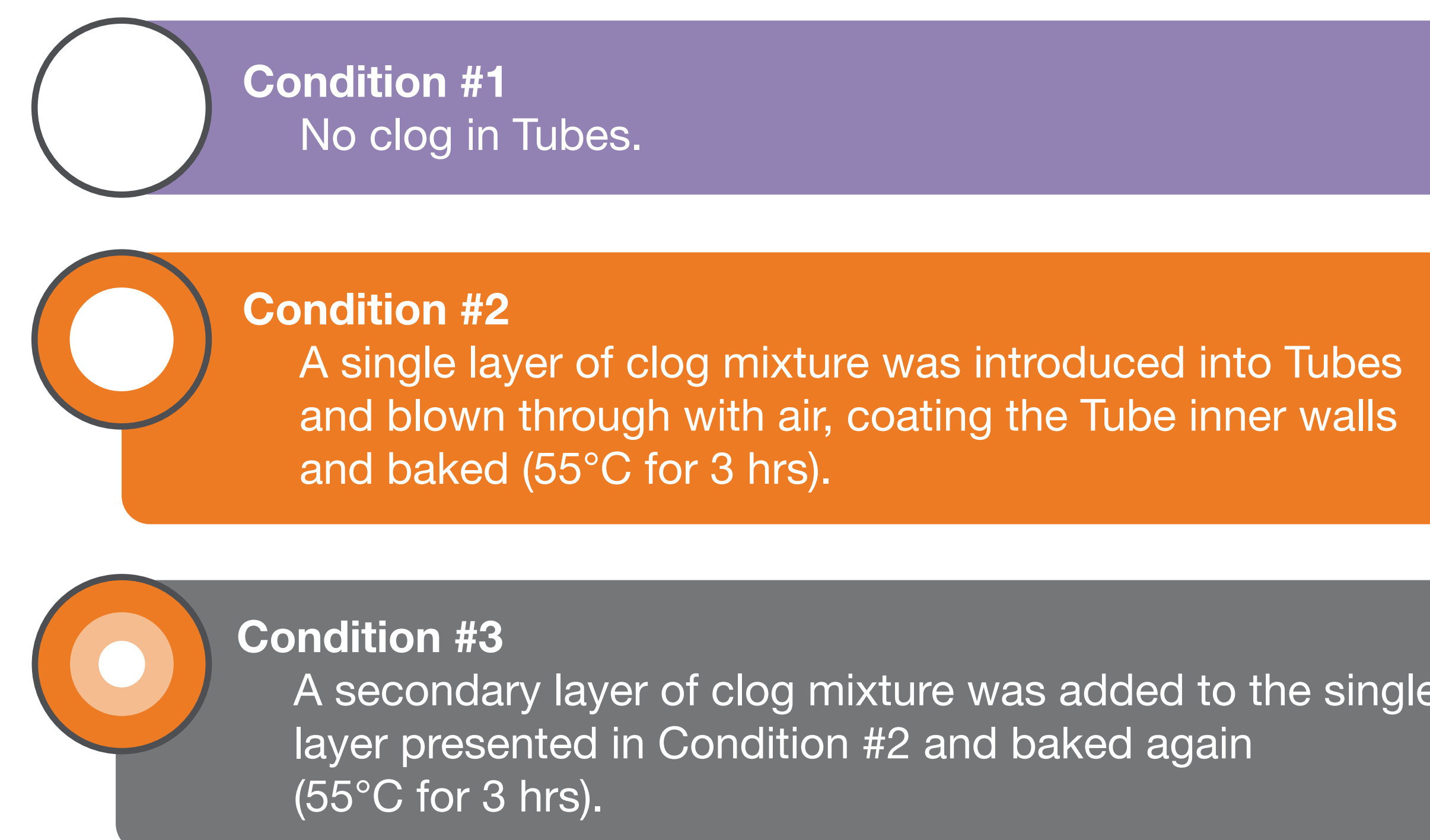


- + **Force Measurement:** A 60-cc syringe was filled with air to the 50-cc mark, and the plunger depressed to the 10-cc mark using a force gauge (Fig 1).



Figure 1

### Results

- + Peak mean force (Newtons [N]) required to push air via the syringe significantly differed due to Tube Condition (Fig 2; Two-way ANOVA, Tukey's Multiple comparison test, different variables =p<0.01).

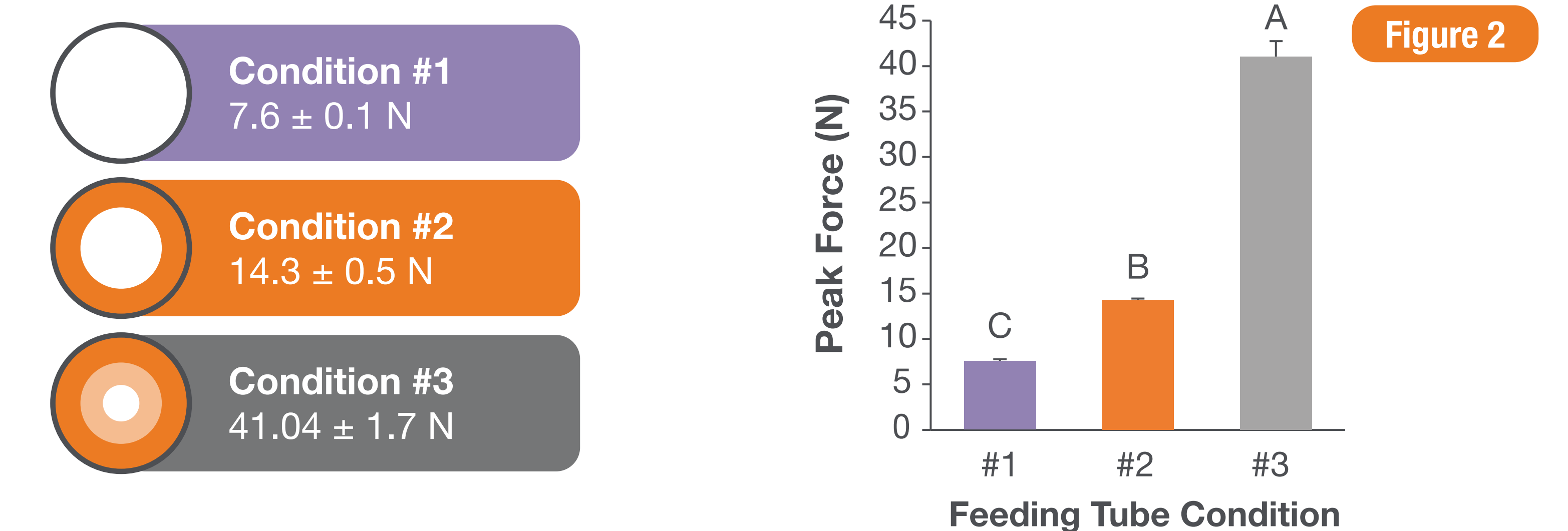

*While these measurements demonstrate an increasing amount of required force, what does that feel like in practice?*

- + Condition #1: Flushed without experiencing any major resistance.
- + Condition #2: Flushed almost as easily as Condition #1; however, the required force being nearly double. Suggesting that Tube clogs, or material build-up on the inner walls, may begin forming before becoming noticeable to healthcare practitioners.
- + Condition #3: Due to increased blockage, attempts to flush air through the Tubes proved the most difficult, requiring additional force as the syringe plunger pushed backwards towards the Operator.
  - Simulated clogs required a force that was nearly **5.5 times** that of the non-clogged Tubes.

### Conclusion

- + Material build-up on the Tube's inner walls may be significant before a healthcare practitioner notices a change in the force required to flush the Tube, thus **they may not realize the Tube's inner diameter has decreased**.
- + By the time a healthcare practitioner notices a change in the required force, the Tube is already substantially clogged, thus **the patient may not achieve their caloric goals**.
- + Using a prophylactic approach prior to noticing changes in syringe force may minimize the risk of material build-up and maintain the full Tube inner diameter, thus **enabling uninterrupted and optimal enteral therapy delivery**.

### References & Acknowledgements

1. Marquard SP, Stegall KS. Unclogging feeding tubes with pancreatic enzyme. J Parenter Enteral Nutr. 1990;14(2):198-200.
2. Bourgault AM, Heyland DK, Drover JW, Keefe L, Newman P, Day AG. Prophylactic pancreatic enzymes to reduce feeding tube occlusions. Nutr Clin Pract. 2003;18(5):398-401.
3. Shiram K, Jayanth V, Lakshmi R, George V. Prophylactic Locking of Enteral Feeding Tubes With Pancreatic Enzymes. Journal of Parenteral and Enteral Nutrition. 2012;21(6):353-356.
4. Mundi M.S., Duellman W, Epp L, Davidson J, and Hurt R.T. (2018). Comparison of Gravity Flow Rates Between ENFit and Legacy Feeding Tubes. Journal of Parenteral and Enteral Nutrition, 42: 522-528. <https://doi.org/10.1177/0148607117703959>

This work described was supported by SBIR Award Numbers NSF-0923861 and NIH/NICHD-R44HD065365. The content is solely the responsibility of the authors and does not necessarily represent the official views of the National Science Foundation or the National Institutes of Health.  
Pat. actuatedmedical.com/ip

